# laservision

**IPL AID**  
user manual

# laservision

**Manufacturer:**

LASERVISION USA, L.P.  
595 Phalen Boulevard  
55130 St. Paul, Minnesota  
UNITED STATES

**EU contact:**

LASERVISION GmbH  
Wuerzburger Str. 152  
90766 Fuerth  
GERMANY  
T +49 911 9736-8100  
F +49 911 9736-8199  
E info@lvg.com  
I uvex-laservision.de

**Notified Body 0196**

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### Directions for Use, Instructions, and Warnings

IPL eye shields are disposable, single-use eye patches to be used during Intense Pulsed Light (IPL) or microdermabrasion treatments. Before using, please read all the general information, instructions, and warnings below. Failure to follow these instructions may result in injury. The IPL eye shields are designed for the medical market. This product is in conformance with the provisions of the standard EN ISO 12609-1:2025.

### Application and safety instructions

- Use in accordance with applicable health and safety regulations. It is the user's responsibility to ensure the IPL eye shields are appropriate protection for the IPL source. If you have questions, consult the manufacturer.
- Wear the eye protector for the whole duration of exposure to hazards. Stop the procedure immediately if the eye protector becomes damaged. It is the user's responsibility to make sure that the shields are correctly placed over the orbital eye area, there are no gaps, and sticking correctly.
- These eye shields are intended to protect against accidental and temporary exposure to IPL exposure. The eye shield is not intended to protect against direct, continual, multiple IPL exposures.
- The eye shield is for patients only. The VLT (Visible Light Transmission) is 0%.
- Eye shields should not be used on or close to open wounds.
- Eye shields are intended for single use only. They are not to be cleaned or used more than once.
- Eye shields are latex free. The material in contact with skin may cause allergic reactions.
- Do not modify this product without prior approval from the manufacturer. There are no spare parts.
- For additional information regarding specific data, and declaration of conformity visit <https://lasersafety.com/ce/>

### Instructions for use

- Inspect the shields for any damage. Damage could be but not limited to discoloration, dents, tears and pitting.
- The patient's orbital area must be cleaned and dried before use.
- When placing the eye shield on the patient, instruct the patient to "**Look Up**" than seal the patch under eye. Then tell the patient to "**Close their eye**" and seal the rest of the eye. Make sure the patch is pressed firmly on the patient's entire orbital area.
- Instruct the patient not to not blink or have rapid eye movement, if they do the eye shield may cause gaps. If this happens, stop procedure immediately and reapply the shield. If it does not reapply, use a new shield or stop procedure.
- When done, carefully remove the eye shield with the tab, gently peeling from one side.
- After use, disposal of the product in accordance with local waste disposal regulations.

### Storage & Maintenance

- Keep the product in its original package. Do not put any weight on it.
- Keep it in a dry, fresh and ventilated area, sheltered from sunlight and away from chemical and abrasive substances. The permissible storage temperature is 0 – 35°C.
- For Transportation every inner box must be kept in strong carton.
- The eye shields expire 2 years after date of manufacturing.

### General Explanations

**LVU** (manufacturer ID) Laservision USA, L.P.

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