laservision

LASER AID user manual

laservision

Manufacturer:

LASERVISION USA, L.P.
595 Phalen Boulevard
55130 St. Paul, Minnesota
UNITED STATES

EU contact:

LASERVISION GmbH
Wuerzburger Str. 152
90766 Fuerth
GERMANY
T +49 911 97 36-8100
F +49 911 97 36-8199
E info@lvg.com
I uvex-laservision.de

Notified Body 0196

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Directions for Use, Instructions, and Warnings

Laser eye shields are disposable, single-use eye patches to be used during laser treatments. Before using the eye shields, please read all the general information, instructions, and warnings below. Failure to follow these instructions may result in injury. The laser eye shields are designed for the medical, dental, and industrial laser market. This product is in conformance with the provisions of the PPE Regulation "Personal Protective Equipment Regulations" PPE Regulation (EU) 2016/425 and with the harmonized standard EN 207:2017.

Application and safety instructions

- For recommendations regarding the use of laser safety eyewear, refer to the corresponding standards. To ensure that the correct laser safety eyewear is selected, always consult the laser safety officer and perform a risk assessment.
- Users shall never look directly at a laser beam.
- The limit values and endurance tests are based on a maximum duration of 5 seconds.
- Laser eye shields are certified to be used to shield the patient's eyes from laser radiation during laser procedures. They are intended to protect against accidental and temporary exposure to laser exposure. The eyewear is not intended to protect against direct, continual, multiple laser exposures.
- Laser eyewear must be worn by anyone within the nominal hazard zone of that laser. Users are at risk of exposure to laser radiation due to reflection from reflective parts (including eye-protectors and filters), tilting or misalignment of optical components. Laser beams that accidentally reflect of objects can cause damage.
- Wear the eye protector for the whole duration of exposure to hazards. Stop the procedure immediately if the eye protector becomes damaged. It is the user's responsibility to make sure that the eye shields are correctly placed over the orbital eye area, there are no gaps, and sticking correctly.
- The laser radiation eyewear is for patients only. The VLT (Visible Light Transmission) is 0%.
- Eye shields should not be used on or close to open wounds.
- Eye shields are intended for single use only. They are not to be cleaned or used more than once.
- Eye shields are latex free. The material in contact with skin may cause allergic reactions.
- Do not modify this product without prior approval from the manufacturer. There are no spare parts.
- For additional information regarding specific data, and declaration of conformity visit <https://lasersafety.com/ce/>

Instructions for use

- Consult the applicable standards for laser in use to determine necessary wavelength protection.
- Check marking on box to confirm laser absorption is at the minimum determined by the laser manufacturer.
- Inspect the shields for any damage. Damage could be but not limited to discoloration, dents, tears and pitting.
- The patient's orbital area must be cleaned and dried before use.
- When placing the eye patch on the patient, instruct the patient to "Look Up" than seal the patch under eye. Then tell the patient to "Close their eye" and seal the rest of the eye. Make sure the patch is pressed firmly on the patient's entire orbital area.
- Instruct the patient not to blink or have rapid eye movement, if they do the eye patch may cause gaps. If this happens, stop procedure immediately and reapply the patch. If it does not reapply, use a new patch or stop procedure.
- When done, carefully remove the eye patch with the tab, gently peeling from one side.
- After use, disposal of the product in accordance with local waste disposal regulations.

Storage & Maintenance

- Keep the product in its original package. Do not put any weight on it.
- Keep it in a dry, fresh and ventilated area, sheltered from sunlight and away from chemical and abrasive substances. The permissible storage temperature is 0 – 35°C.
- For Transportation every inner box must be kept in strong carton.
- If the disposable eye shields have been damaged, or undergone color change, they shall not be used and should be replaced.
- The eye shields expire 2 years after date of manufacturing.

Explanation of the symbols used in the marking

316-1400 D LB5 + IR LB7 + M LB7Y LVU CE

1401-11000 DI LB3 LVU CE

316-1400	indicates the wavelength range in nanometers
D (test condition for laser type)	continuous wave laser
I (test condition for laser type)	pulsed laser, >10-6 to 0.25s
R (test condition for laser type)	Q-switch pulsed laser, pulse length >10-9 to 10-6s
M (test condition for laser type)	Mode-coupled pulse laser, pulse length <10-9s
LB	indicates the protection level (LB1 – LB10) and lower limit for the optical density
LVU	(manufacturer ID) Laservision USA, L.P.
CE	Sign for European Conformity
OD	The symbol "OD" refers to Optical Density; it may be engraved on the filter, frame, or eye-protector. The corresponding number range refers to the range of wavelengths protected at that Optical Density.

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